

respond
making life better


HiLINE



support wear

**specialists in stoma
and continence care**

NHS
Providing NHS services
an eakin company 

support wear for you

Our range of support garments are designed to provide abdominal support following bowel surgery. They can be used for hernia and hernia symptoms support, or as a preventative measure against hernias during physical activity post-op.

HiLINE garments provide comfort and discretion under clothing, whilst maintaining a high level of support. The range provides comprehensive solutions to any situation that requires the need to use a support wear product. We offer a wide range of products in different styles, depths and colours, with the option to customise all HiLINE garments with a hole over the stoma, if required, giving you more freedom and choice so that the garments can blend seamlessly into your lifestyle.

The HiLINE support wear range are suitable for conditions such as:

- ✓ Ileostomy
- ✓ Post-abdominal surgery
- ✓ Colostomy
- ✓ Hernia prevention and support
- ✓ Urostomy
- ✓ Incontinence

our levels explained

Our wide range of support products are grouped into 3 level types; level 1 for light support, level 2 for medium support, and level 3 for firm support, making it easier for you to choose what garment you need, for the level of support that you require.



light support - level 1

Light support garments are suitable immediately and up until 3 months post-surgery. They support the abdomen and stoma site whilst carrying out gentle core exercises and walking.



medium support - level 2

Medium support garments are suitable for active stoma or hernia patients. Wearing medium support enables the wearer to be active at work, in sports and moderate lifting whilst protecting the stoma (not recommended for intense activity).



firm support - level 3

Firm support garments protect the stoma or hernia and allow the wearer to be more active. These garments should only be worn when active and are ideal for intense activity such as heavy lifting, manual labour, gardening and energetic/extreme sports.

"Thank you so much for an excellent service, I would recommend it to anyone"

Keith, Llanelli



respond
making life better

**AWARD-WINNING
PRESCRIPTION
DISPENSING SERVICE**



"Respond took over the reins with my Ileostomy equipment and have been instrumental in my recovery"

Paul, Perth



"Very helpful, and very understanding"

Lauraine, Kidwelly



**Professional, friendly
and reliable.**

**Everyday, we make
life better.**

A service to smile about

99.5%

of our customers say we are **very good** for holding stock of all products

99.7%

of our customers say the quality of our service is **excellent or very good**

99.7%

of our customers would recommend **us**

Get in touch to find out more about our service



0800 220 300



0800 028 6848 (N.Ireland)



respond.co.uk

**specialists in stoma
& continence care**

Data on file

NHS

Providing NHS services

an eakin company

light support – level 1

The light support underwear is suitable for everyday use replacing regular underwear and can be worn under a waistband for extra support.

They are very comfortable to wear and provide light support without being restrictive and will not affect a stoma pouch from functioning. Ideal for those who require gentle support following surgery or for everyday use.



light control support unisex garments

- ✓ High waisted
- ✓ 100% stretch
- ✓ Seam free
- ✓ MRSA resistant

unisex garments are designed to provide light support following surgery and are available in white, black or beige colour choices.

“In all the years I have had a colostomy, I have never had a garment that doesn’t put pressure on my stoma, yet supports me totally, and feels so light next to my body as if the garment is not there. They are brilliant to wear.”

Mrs C. P. (London)

“CoolCOMFORT make a vast difference to my life. They hold the bag securely and make me feel more comfortable because I feel safe.”

Mr R. A. (Stoke on Trent)



Boxer



Brief



Support belt

- ✓ Choice of boxer, brief or belt
- ✓ Underwear has a high waistband to cover a stoma pouch and comes with a 180° stretch
- ✓ Hydrophilic – wipes away sweat and moisture
- ✓ Seam-free – no rubbing/irritation
- ✓ Provides comfort, security and peace of mind
- ✓ Can be worn day and night

For sizes, codes and to order please contact us:



phone

0800 220 300
0800 028 6848 (N.Ireland)



online

respond.co.uk



email

hello@respond.co.uk

medium support – level 2

Level 2 support wear garments are ideal for everyday use including sport, work and leisure activities.

Supportive yet comfortable to wear, our level 2 garments are designed to support and relieve hernia symptoms, to protect hernia repairs, and to reduce the risk of developing a hernia post-surgery.



medium control ostomy female pantie brief

This garment gives medium support whilst still retaining the wearer's femininity. The front lining is created from a stain resistant, cotton/nylon mix open weave lace front. Available in black or white colour choices. The cross over waist band gives extra comfort when sitting and helps prevent the garment from rolling over at the top. These garments are designed with a classic leg fitting.

- ✓ Provides medium support to relieve hernia symptoms, protects hernia repairs and helps prevent any recurrence
- ✓ Promotes security, confidence and peace of mind
- ✓ Cross over waist band gives extra comfort and prevents the garment from rolling



Res 2 white lace



Res 2 Black lace

Colour	Size	Order code
White or Black	8-30	Res 2

medium unisex support belts

This lightweight belt has a strong cotton front, lined with stain resistant soft peached fabric and has been designed to support the areas that need it most. The back is made up of cushioned elastic, which helps to keep the skin cool and may help with lower back support. These garments are soft on the skin and comfortable to wear, whilst providing solid support during physical activity. The Res 4 belts have an extra 8cms of depth compared to the Res 3. Those with a larger hernia or a longer body will benefit from the additional depth that this style provides.

- ✓ Provides medium support to protect an existing hernia and preventative support to reduce the risk of developing a hernia
- ✓ Designed with a cotton front support section and lined with a stain resistant very soft peached fabric
- ✓ The back is made from cushioned elastic to keep the skin cool
- ✓ Adjustable Velcro® Brand fastening

Available sizes S-XXL

Res 3 (13cm Deep)



White



Black



Beige

Res 4 (21cm Deep)



White



Black



Beige

Depth	Colour	Size	Order code
13cm	White/Black/Beige	S-XXL	Res 3
21cm	White/Black/Beige	S-XXL	Res 4

Please specify choice of colour and size of garment when ordering.


*Bespoke sizes available upon request.

seamless 2 way stretch unisex colostomy belt


Made from a seamless strong knitted fabric with two way stretch, this support belt is ideal for people who have dexterity issues or difficulty with fastenings. The seamless design gives better conformity and is less obtrusive under close fitting clothing. The low fabric weight is a breakthrough in technology for the level of compression achieved. Very lightweight and exceptionally strong, yet comfortable to wear, it is also suitable for night time use. The Res 8 option provides the same style details, features and fabric as the Res 7 above, but has an extra 7cms in depth. If you need a deeper garment you must order this item with the correct code, Res 8, on your prescription.

- ✓ Provides medium to firm support to protect an existing hernia or preventative support to reduce the risk of developing a hernia
- ✓ Made from a seamless strong knitted fabric

Res 7 (26cm Deep)

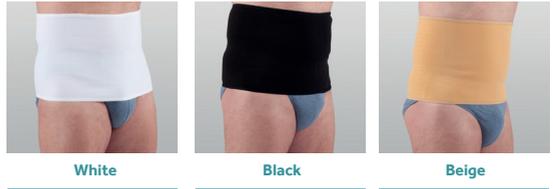

Res 8 (33cm Deep)

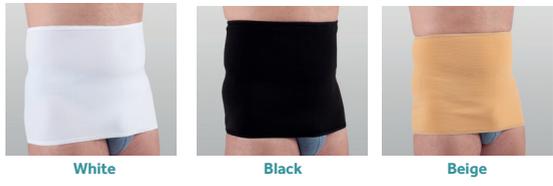

Depth	Colour	Size	Order code
26cm	White/Black/Beige	S-XXL	Res 7
33cm	White/Black/Beige	S-XXL	Res 8

Please specify choice of colour and size of garment when ordering.
 *Bespoke sizes available on request.

Activ belt

The HiLINE Activ range is available in 4 plain unisex colours and in sizes S, M, L, XL and XXL.

*Bespoke sizes available upon request.

Did you know?

our HiLINE Activ support belts completely hide the stoma bag and have a leak proof inner lining



Turquoise



Black



Navy



Beige

Get in touch today



phone

0800 220 300
0800 028 6848 (N.Ireland)



online

respond.co.uk



email

hello@respond.co.uk

EZ-Wrap hernia support belt

The HiLINE EZ-Wrap unisex hernia support belt is a brand new design which provides medium/ firm level hernia support or prophylactic support and is also ideal for those with dexterity issues or who normally struggle to fasten a standard Velcro® Brand support belt.

The EZ-Wrap belt is designed with a side zip pocket and additional lugs for ease of application and an anti-roll grip effect waistband.

- ✓ Medium/Firm Support Wear
- ✓ Anti-roll grip effect waistband
- ✓ Zip pocket for ease of application
- ✓ Additional lugs for ease of application
- ✓ Reinforced support around stoma



White, black and beige - available with a hole option

When ordering please remember to specify colour and size required. If you need a hole in your belt you must supply your pouch code number (baseplate code also required if using a 2 piece system).

Depth	Colour	Size	Order code
26cm	White/Black/Beige	S-XXL	Res 11
20cm	White/Black/Beige	S-XXL	Res 11a
15cm	White/Black/Beige	S-XXL	Res 11b

with customised hole

Depth	Colour	Size	Order code
26cm	White/Black/Beige	S-XXL	Res 11h
20cm	White/Black/Beige	S-XXL	Res 11ah
15cm	White/Black/Beige	S-XXL	Res 11bh

firm support – level 3

Level 3 support garments are designed for those with larger hernias who require stronger support or for those who are participating in more strenuous activities and require something more supportive as a preventative measure.

Strong and reliable, but comfortable for everyday use including leisure and work activities such as sport, exercising, lifting and gardening.



ostomy female pantie brief

Manufactured in the highest quality satin or cotton damask, this garment gives a medium to firm support whilst retaining the wearer's femininity. The front lining is created from a stain resistant, soft peached cotton/polyester mix. The gusset is lined with cotton jersey for comfort. Available in black or white colour choices. The cross over waist band gives extra comfort when sitting and helps prevent the garment from rolling over at the top. These garments are designed with a high leg fitting.

- ✓ Provides medium to firm support to relieve hernia symptoms, protects hernia repairs and helps prevent any recurrence
- ✓ Cross over waist band gives extra comfort and prevents garment from rolling
- ✓ Improves appearance and promotes security, confidence and peace of mind

Res 1 (satin or cotton)



Satin white



Satin black



Cotton white



Cotton black




Cotton pattern detail

Colour	Size	Order code
White/Black	8-30	Res 1

breathable stretch unisex ostomy belt


This stretch elastic fabric constructed of a cotton/nylon/ rubber mix will give firm support to a hernia and the lower back. The open weave allows the skin to breathe reducing heat and improving comfort levels, therefore proving vastly beneficial for stoma pouch users and an ideal choice for all ostomates. The breathable belt comes in a choice of 3 depths, each one provides the same style details, features and fabrics but the choice of depths allows those who prefer a narrower fit or who have a longer body, the option of a belt depth to suit all body requirements.

Res 6 (21cm deep)

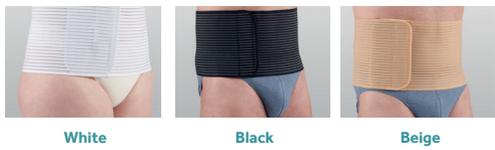


- ✓ Provides firm support to protect an existing hernia or preventative support to reduce the risk of developing a hernia
- ✓ The open weave fabric allows the skin to breathe

Res 6a (18cm Deep)

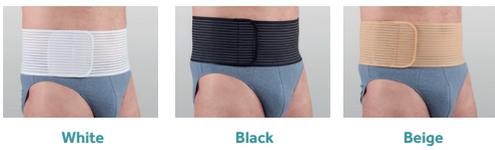

Res 6b (13.5cm deep)

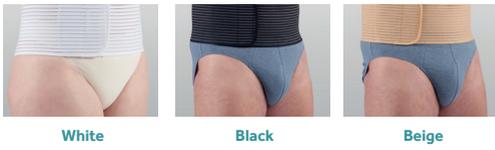

Depth	Colour	Size	Order code
21cm	White/Black/Beige	S-XXL	Res 6
18cm	White/Black/Beige	S-XXL	Res 6a
13.5cm	White/Black/Beige	S-XXL	Res 6b

Please specify choice of colour and size of garment when ordering.

*Bespoke sizes available on request

SHIELD

stoma protector

A lightweight, moulded shield designed to help protect the stoma from direct contact, SHIELD has been shaped to allow the stoma pouch to function during wear and can be worn discreetly under any type of clothing. The SHIELD stoma protector is held in place by a detachable, strong, adjustable elastic belt (ordered separately).

SHIELD offers security and protection during sporting activities, gardening, whilst at work and also to prevent car seat belts from rubbing against the stoma.

- ✓ Protects the stoma from direct contact
- ✓ One size shield fits all
- ✓ Ideal for sport, work, gardening and whilst driving etc.
- ✓ Lightweight
- ✓ Does not affect pouch functionality
- ✓ Adjustable belt
- ✓ Quick and easy to put on and take off
- ✓ Specially designed for ostomates



Res50 SHIELD and Res40/45 Belt

Product	Colour	Size	Order code
SHIELD stoma protector without belt	White	ONE	Res 50
SHIELD BELT	White	S/M (40-85cm)	Res 40
SHIELD BELT	White	L/XL (66-124cm)	Res 45

Please ensure you order the correct size belt with your SHIELD.

Order now



phone

0800 220 300

0800 028 6848 (N.Ireland)



online


respond.co.uk




email

hello@respond.co.uk

Additional support for you and your nurse

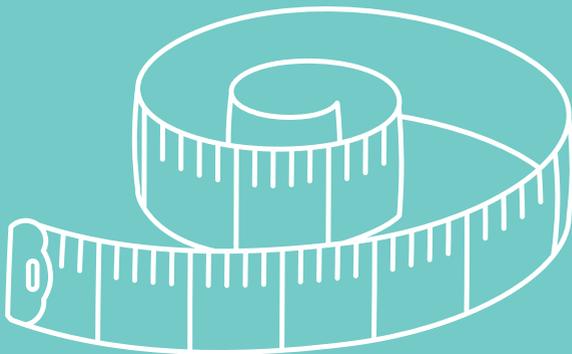

Minimise the risk of developing a hernia by following these simple exercises

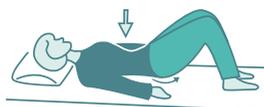

Pelvic tilting

Lie on your back with your knees bent and feet flat on the bed. Pull your lower tummy in, tilt your bottom upwards slightly whilst pressing the middle of your back into the bed. Try to breathe out as you are doing this and hold the position for two seconds. Let go slowly.



Knee rolling

Lie on your back with your knees bent and feet flat on the bed. Pull your tummy muscles in. Keeping your knees together, slowly roll them from side to side. A little way at first, then gradually increasing as far as is comfortable.

Please note that the above exercises are a guide to introduce gentle exercises to help strengthen the core. Your abilities will vary depending on surgery type and before starting any new exercises please consult your Consultant or Stoma Care Nurse.

size chart guide

Level 1



Size chart

Size	S/M	L/XL	XXL
to fit hips	87-96.5cm	99-117cm	119.5-127cm
to fit hips	34-38 inch	39-46 inch	47-50 inch

If you would like to order a support garment but haven't been fitted for one before, give us a call for your **free** appointment!



Size chart for Res 3, 4, 6, 6a, 6b, 7, 8 and HiLINE Activ and EZ-Wrap belts

When selecting your belt size below, remember to measure around your body in line with the stoma.

Small	Medium	Large	Extra Large	XX Large
78.5-86.5cm	89-96.5cm	99-107cm	109-117cm	119.5-127cm
31-34 inch	35-38 inch	39-42 inch	43-46 inch	47-50 inch



Size chart for Res 1 and Res 2 ostomy pantie briefs

Size	8	10	12	14	16	18	20	22	24	26	28	30
to fit hips cm	87	90	95	100	105	110	115	121	126	131	136	141
to fit hips inch	34.5	35.5	37.5	39.5	41.5	43.5	45.5	47.5	49.5	51.5	53.5	55.5

*Bespoke sizes available upon request

HiLINE Active

prescriptions order codes



Order codes for **Left Sided** stoma with a **Standard** size hole required (2¼ - 2¾ inches)

COLOURS AVAILABLE	STOMA LOCATION	HOLE DIAMETER	ORDER CODE
Black	Left	A	Res 9al
White	Left	A	Res 9al
Turquoise	Left	A	Res 9al
Beige	Left	A	Res 9al



Order codes for **Right Sided** stoma with a **Standard** size hole required (2¼ - 2¾ inches)

COLOURS AVAILABLE	STOMA LOCATION	HOLE DIAMETER	ORDER CODE
Black	Right	A	Res 9ar
White	Right	A	Res 9ar
Turquoise	Right	A	Res 9ar
Beige	Right	A	Res 9ar



Order codes for **Left Sided** stoma with a **Large** size hole required (2¾ - 3¼ inches)

COLOURS AVAILABLE	STOMA LOCATION	HOLE DIAMETER	ORDER CODE
Black	Left	B	Res 9bl
White	Left	B	Res 9bl
Turquoise	Left	B	Res 9bl
Beige	Left	B	Res 9bl



Order codes for **Right Sided** stoma with a **Large** size hole required (2¾ - 3¼ inches)

COLOURS AVAILABLE	STOMA LOCATION	HOLE DIAMETER	ORDER CODE
Black	Right	B	Res 9br
White	Right	B	Res 9br
Turquoise	Right	B	Res 9br
Beige	Right	B	Res 9br



Order codes for **Left Sided** stoma with a **Bespoke** size hole require (less than 2¼ inches or more than 3¼ inches)

COLOURS AVAILABLE	STOMA LOCATION	HOLE DIAMETER	ORDER CODE
Black	Left	C	Res 9cl
White	Left	C	Res 9cl
Turquoise	Left	C	Res 9cl
Beige	Left	C	Res 9cl



Order codes for **Right Sided** stoma with a **Bespoke** size hole required (less than 2¼ inches or more than 3¼ inches)

COLOURS AVAILABLE	STOMA LOCATION	HOLE DIAMETER	ORDER CODE
Black	Right	C	Res 9cr
White	Right	C	Res 9cr
Turquoise	Right	C	Res 9cr
Beige	Right	C	Res 9cr

To order please get in touch with our friendly customer services team



phone

0800 220 300

0800 028 6848 (N.Ireland)



online

respond.co.uk



email

hello@respond.co.uk

our expertise

We are a **UK company** proud to provide quality garments to this specialised market. With a wealth of experience in the field of hernia and stoma prevention, we have the experience and knowledge to help you and your patients with support garment requirements and fittings.

All of our support garments are **handmade** to order by a specialist team in the **UK**.

We have the following teams available who have all been trained in measuring and fitting garments.

*Dedicated team
of nurses*

Dedicated fitters

*Dedicated Area
Managers*

we offer



Clinics – we can facilitate a garment clinic for you, or alongside your regular stoma clinics in your hospital, or in one of our dedicated treatment rooms where we can see individuals or hold regular clinics.



Home visits - for house bound patients .

If you are interested in any of the above services please get in touch with your local representative or call our friendly customer services team on

 **0800 220 300**

 **0800 028 6848** (N.Ireland)

 **hello@respond.co.uk**

 **respond.co.uk** to view the full range



continence and stoma swimwear

We take pride in our expertly designed and UK manufactured swimwear.

Continence

- Suitable for both faecal and urinary incontinence.
- Each garment has a secure and water-tight integral pant which will contain any accidents whilst swimming.

Stoma

- Designed to give the wearer more confidence when swimming.
- Discreet internal pocket to hold the stoma pouch in place.
- Mens swimshorts are 2" higher in the waist.
- Ladies features a flattering ruched panel for added discretion.



For a full price list and to buy online please get in touch today.



phone

0800 220 300

0800 028 6848 (N.Ireland)



online

respond.co.uk



email

hello@respond.co.uk



Providing NHS services

an eakin company 



Connect magazine




The complete range of products



Ostomy and continence swimwear




Travel guide advice booklet

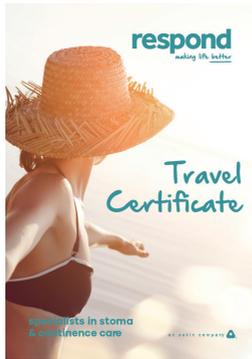


Travel Certificates




Sex & intimacy advice

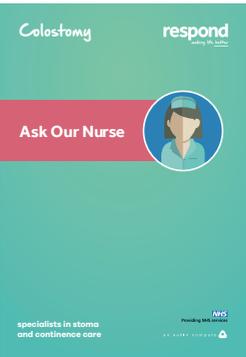

Support for managing your colostomy

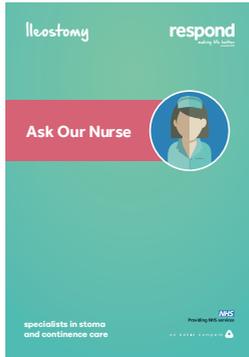

Support for managing your ileostomy

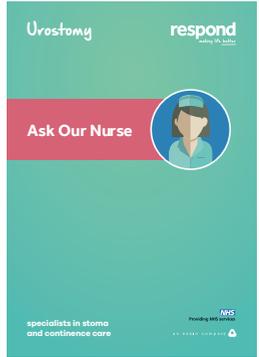

Support for managing your urostomy

Please send the above ticked brochures to

Name

Address

Post Code

Telephone Email

Please detach and send to:
 Freepost RTSZ-SLYE-EBTL, Respond Healthcare Ltd, Greypoint, Cardiff Business Park, Parc Ty Glas, CARDIFF, CF14 5WF
 Data Protection Act 1998

The information supplied on this form will be retained by Eakin Healthcare Group on a database. The data will not be disclosed to any external sources. Please indicate here if you wish to hear of further products and services available from Eakin Healthcare Group SG

**making
life
better**

0800 220 300

0800 028 6848 N. Ireland

Respond Healthcare Ltd

hello@respond.co.uk

respond.co.uk

f respondltd

🐦 @respondltd

Useful Contacts

Colostomy UK

info@colostomyuk.org

0800 328 4257

colostomyuk.org

IA (Ileostomy & Internal Pouch Association)

info@iasupport.org

0800 018 4724

iasupport.org

Urostomy Association

info@urostomyassociation.org.uk

01386 430 140

urostomyassociation.org.uk

